


Sway Medical


Baseline Testing





Scan QR code to download
the Sway Medical App

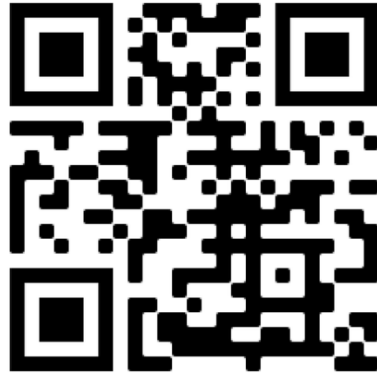

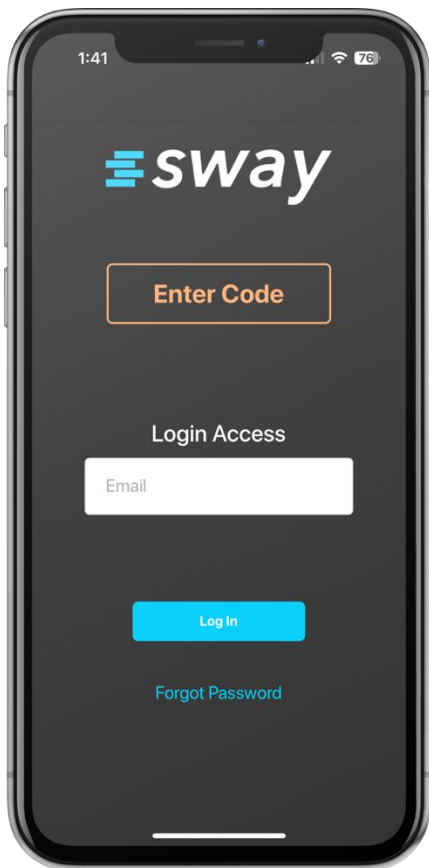

- Put your phone on **DO NOT DISTURB**
- Close out all other apps
- Disable power mode or rise to wake
- Enable rotate screen
- Phone battery more than 10%



Enter the 5-digit
code at the TOP
of your screen

43R7M





Create a Profile

- Basic information
- Education
- Medical

Click “Finish” once all fields are filled out.


The image shows a smartphone screen displaying the 'Manage Profile' app interface. At the top, the status bar shows the time 10:37, signal strength, Wi-Fi, and battery icons. Below the status bar, there are two buttons: 'Cancel' on the left and 'Manage Profile' on the right. The main content area features a blue icon of a person with their hands clasped. Below the icon, the text reads 'Nice to meet you, we're Sway!' followed by a smaller line: 'We look forward to getting to know you. Let's get your profile set up so we can get started.' There are four small blue dots below this text, with the first one being larger and filled. The section is titled 'Profile Information' and contains two text input fields. The first field is labeled 'First Name' and has the placeholder text 'ex. John'. The second field is labeled 'Last Name' and has the placeholder text 'ex. Smith'. At the bottom of the form, there is a blue button with the text 'NEXT: BASIC INFORMATION'.



Click Hardware Verification!

Then start the symptoms checklist


- Complete the symptoms based on how you feel TODAY

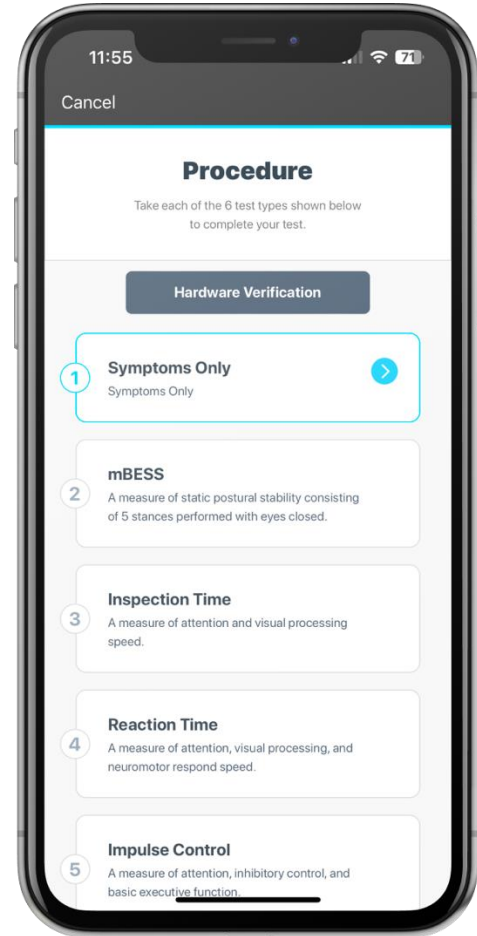



Follow the instructions on the screen!

- For each test you will complete a practice trial then 2-3 actual trials
- After you complete the balance test – sit down to complete the rest
- Check in with me before you leave the testing site

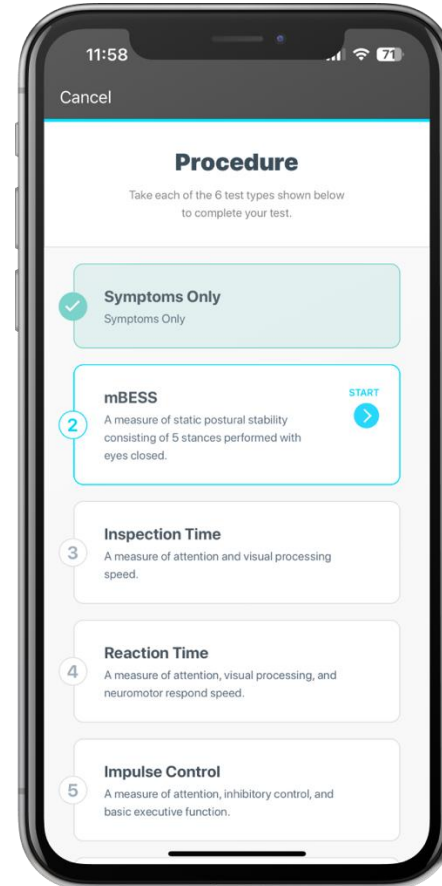



Your team's
code is

43R7M

